---

Neu ab:  
New as of:  
Nouveau depuis:  
Nuevo a partir de:

---

06.2019

 Dentsply  
Sirona

# Carl, Paul

Gebrauchsanweisung Zahnärztlicher Arbeitsstuhl  
Operating Instructions for Dentist's Working Stool  
Notice d'utilisation - Siège de travail dentaire  
Manual del operador de la silla odontológica

Deutsch / English / Français / Español

# Carl, Paul

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---

# Carl, Paul

Operating Instructions for Dentist's Working Stool

English

## Table of contents

<b>1</b>	General data .....	<b>3</b>
1.1	Dear Customer,.....	3
1.2	Contact data.....	3
1.3	Structure of the document.....	4
1.3.1	Identification of the danger levels .....	4
1.3.2	Formats and symbols used.....	4
<b>2</b>	Safety information .....	<b>5</b>
2.1	Modifications to the product .....	5
2.2	Accessories.....	5
2.3	Maximum load capacity .....	5
<b>3</b>	Product overview .....	<b>6</b>
3.1	Carl working stool .....	6
3.2	Paul working stool.....	7
<b>4</b>	Operation .....	<b>8</b>
4.1	Height adjustment with manual release .....	8
4.2	Backrest .....	9
<b>5</b>	Care and cleaning.....	<b>10</b>
5.1	Care, cleaning agents, and disinfectants .....	10
5.2	Care and cleaning.....	11
<b>6</b>	Technical data .....	<b>12</b>
6.1	Carl working stool .....	12
6.2	Paul working stool.....	13

# 1 General data

## 1.1 Dear Customer,

Thank you for purchasing your Carl / Paul working stool from Dentsply Sirona.

The technical documentation supplied is also part of the product. You should always keep this documentation within reach.

These Operating Instructions show the various models of the working stool.

Care and cleaning should be carried out based on the corresponding instructions.

It is important to observe all safety information to prevent personal injury and material damage.

Your Dentsply Sirona team

## 1.2 Contact data

### Customer service center

For technical questions, use the contact form on the internet at [www.sirona.com](http://www.sirona.com). Follow the menu items "*CONTACT*" / "*Customer Service Center*" on the navigation bar and click on the "*CONTACT FORM FOR TECHNICAL QUESTIONS*" button.

Sirona Dental Systems GmbH  
Fabrikstrasse 31  
64625 Bensheim  
Germany

Phone: +49 (0) 6251/16-0  
Fax: +49 (0) 6251/16-2591  
E-mail: [contact@sirona.com](mailto:contact@sirona.com)  
[www.sirona.com](http://www.sirona.com)

## 1.3 Structure of the document

### 1.3.1 Identification of the danger levels

To prevent personal injury and material damage, please observe the warning and safety information provided in these operating instructions. Such information is highlighted as follows:

 <b>DANGER</b>
An imminent danger that could result in serious bodily injury or death.

 <b>WARNING</b>
A possibly dangerous situation that could result in serious bodily injury or death.

 <b>CAUTION</b>
A possibly dangerous situation that could result in slight bodily injury.

<b>NOTE</b>
A possibly harmful situation which could lead to damage of the product or an object in its environment.

<b>IMPORTANT</b>
Application instructions and other important information.

**Tip:** Information for simplifying work.

### 1.3.2 Formats and symbols used

The formats and symbols used in this document have the following meaning:

<ul style="list-style-type: none"> <li>✓ Prerequisite</li> <li>1. First action step</li> <li>2. Second action step</li> <li>or</li> <li>➤ Alternative action</li> <li>↔ Result</li> <li>➤ Individual action step</li> </ul>	Requests you to do something.
See "Formats and symbols used [→ 4]"	Identifies a reference to another text passage and specifies its page number.
• List	Designates a list.
"Command / menu item"	Indicates commands / menu items or quotations.

## 2 Safety information

### 2.1 Modifications to the product

Modifications to this product which may affect the safety of the operator, patients or third parties are prohibited by law!

### 2.2 Accessories

In order to ensure product safety, this device may be operated only with original Dentsply Sirona accessories or third-party accessories expressly approved by Dentsply Sirona. The user is responsible for any damage resulting from the use of non-approved accessories.

### 2.3 Maximum load capacity

Please observe the load capacity of the working chair of **135kg**.

## 3 Product overview

### 3.1 Carl working stool

#### "Manual" model

Height adjustment with manual release

#### "Manual Plus" model

Height adjustment with manual release and foot ring

## 3.2 Paul working stool

- Height adjustment with manual release
- Foot ring (optional)

## 4 Operation

### 4.1 Height adjustment with manual release

#### Carl working stool


The operating ring for manual release is located **underneath the seat**.

- **Raising**

Take the weight of your body off of the seat as much as possible and pull the operating ring upward.

- **Lowering**

Leave the weight of your body on the seat and pull the operating ring upward.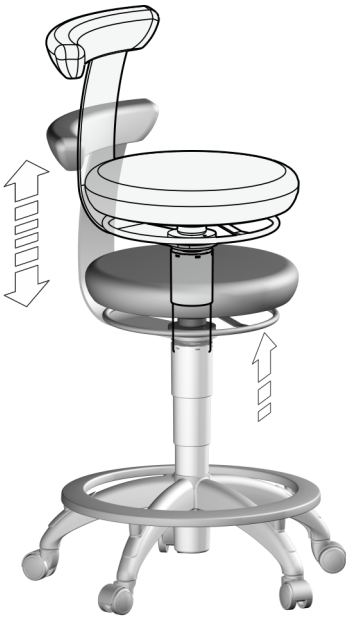

#### Paul working stool


The operating lever for manual release is located **underneath the seat**.

- **Raising**

Take the weight of your body off of the seat as much as possible and pull the operating lever upward.

- **Lowering**

Leave the weight of your body on the seat and pull the operating lever upward.

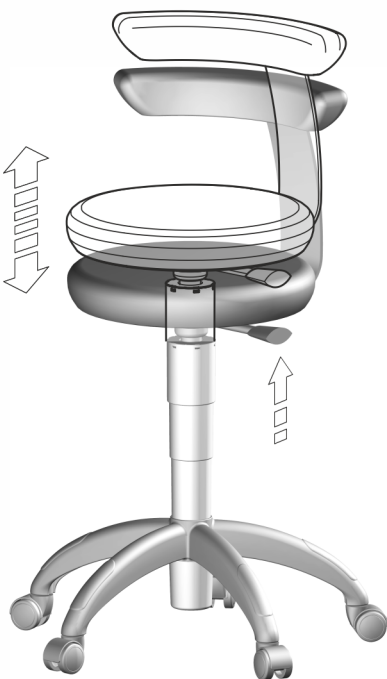## 4.2 Backrest

### Carl working stool

The backrest can be swiveled 360°.

### Paul working stool

The backrest can not be swiveled.

## 5 Care and cleaning

### 5.1 Care, cleaning agents, and disinfectants

#### NOTE

##### Approved care, cleaning, and disinfecting agents

Use only care, cleaning, and disinfecting agents approved by Dentsply Sirona!


A continuously updated list of approved agents can be downloaded from the Internet on the online portal for technical documents. You can reach this portal at the address:  
[www.dentsplysirona.com/manuals](http://www.dentsplysirona.com/manuals)  
.Click on the menu item "*General documents*" and then open the "*Care, cleaning and disinfection agents*" document.

If you do not have access to the internet, please contact your dental depot to order the list (REF 59 70 905).

## 5.2 Care and cleaning


- **Disinfecting**  
Surfaces can be sprayed or wiped with surface disinfectants.
- **Cleaning**  
Remove dirt, grime and disinfectant residue regularly using mild, commercially available cleaning agents.
- **Upholstery**  
Use only media approved by Sirona for upholstery; see "Care, cleaning, and disinfecting agents"!
- **Protection against medicaments**  
Due to their high concentrations and the substances they contain, many medicaments can dissolve, etch, bleach or discolor surfaces. The only way to prevent damage is to wipe off medicaments immediately with a damp cloth and a cleaning agent!
- **Castors**  
Clean the castors of the working stool at regular intervals to prevent damage to the floor.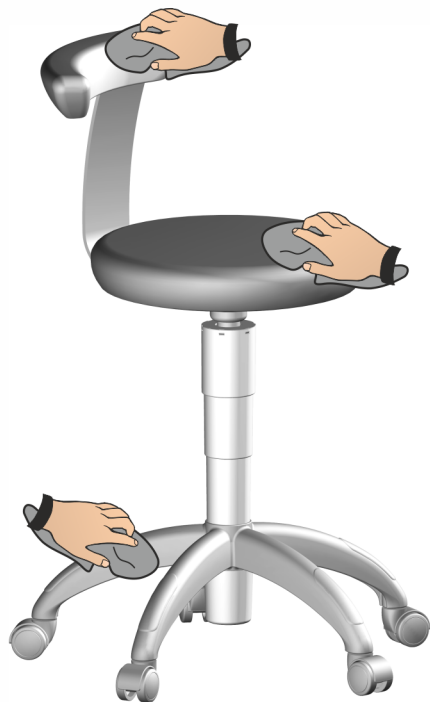


## 6 Technical data

### 6.1 Carl working stool

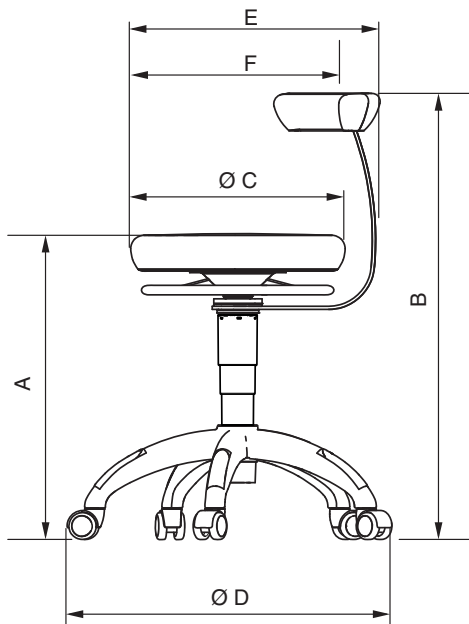

		Dimensions in mm
<b>A</b>	Seat height (acc. to DIN, loaded)	
	Pneumatic spring, short	420-545
	Pneumatic spring, long	525-645
<b>B</b>	Total height (in lowest position, unloaded)	
	Pneumatic spring, short	720
	Pneumatic spring, long	825
<b>C</b>	Seat width	Dia. 400
<b>D</b>	Star base	Dia. 577
<b>E</b>	Total depth	465
<b>F</b>	Seat depth (acc. to DIN, in lordosis support point)	385
<b>G</b>	Backrest height	265
<b>H</b>	Backrest width	407
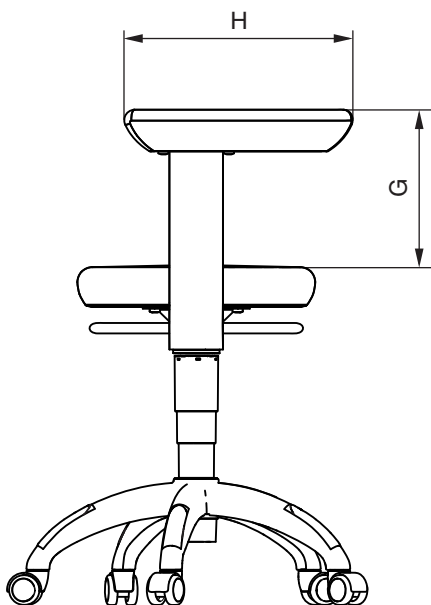
#### Total weight

11.3kg - 12.5kg – suitable for persons weighing up to 135kg

## 6.2 Paul working stool

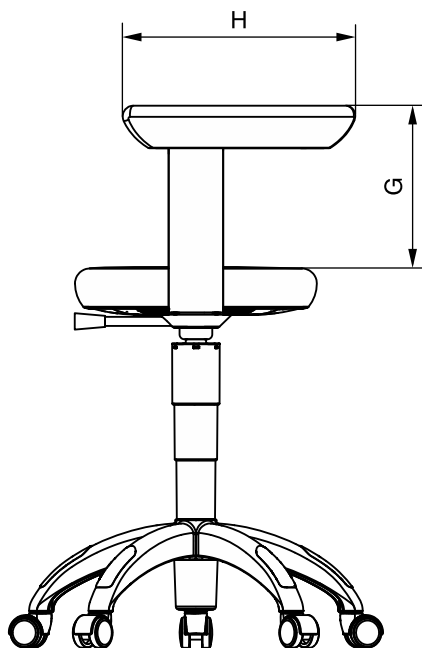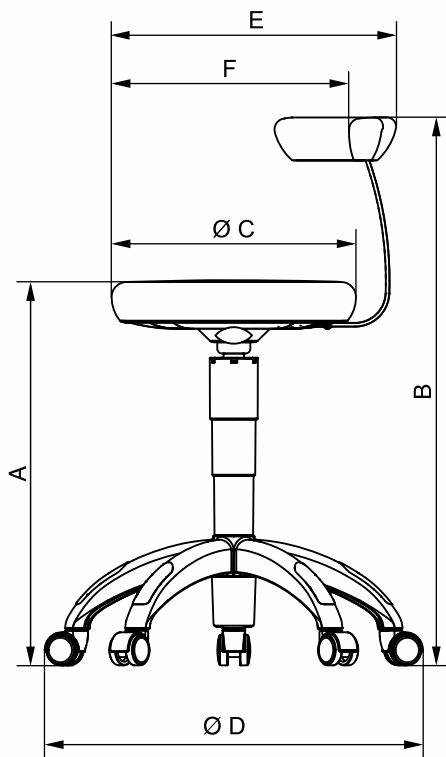		Dimensions in mm
<b>A</b>	Seat height (acc. to DIN, loaded)	
	Pneumatic spring, short	420-545
	Pneumatic spring, long	525-645
<b>B</b>	Total height (in lowest position, unloaded)	
	Pneumatic spring, short	720
	Pneumatic spring, long	825
<b>C</b>	Seat width	Dia. 400
<b>D</b>	Star base	Dia. 577
<b>E</b>	Total depth	465
<b>F</b>	Seat depth (acc. to DIN, in lordosis support point)	385
<b>G</b>	Backrest height	265
<b>H</b>	Backrest width	407

### Total weight

9.3kg - 10.5kg – suitable for persons weighing up to 135kg

---

We reserve the right to make any alterations which may be required due to technical improvements.

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© Sirona Dental Systems GmbH